## SURFACE DISINFECTION CONCENTRATE NON-ALCOHOLIC

**Concentrate for disinfection and cleaning of medical inventory, medical devices and all kinds of surfaces**

- **Aldehyde-free preparation with good cleaning**
- **Perfume-free**

### PRODUCT DESCRIPTION:

CyberTech Surface disinfection concentrate non-alcoholic is an aldehyde-free concentrate for disinfection and cleaning of medical inventory, medical devices and other kinds of surfaces. The alcohol-free formulation is based on modern quaternary ammonium compounds and contains neither aldehydes nor phenols. CyberTech Surface disinfection concentrate non-alcoholic is very userfriendly due to its good cleaning properties and perfume-free formulation. CyberTech Surface disinfection concentrate non-alcoholic can be used as an alcohol-free rapid disinfectant and is suitable for the disinfection and cleaning of acrylic glass.

### AREAS OF APPLICATION:

According to Medical Device Directive (MDD): For cleaning and disinfecting non-invasive medical devices. According to Biocidal Products Regulation (BPR): For cleaning and disinfecting all types of surfaces.

### SPECTRUM OF ACTION:

Bactericidal (incl. MRSA), yeasticidal, virus-inactivating (HBV/HIV/HCV/BVDV/Vaccinia/Papova viruses): 2,5 % (25g/l) – 15 min.; 2 % (20g/l) – 30 min.; 1 % (10g/l) – 60 min.

### CONCENTRATION:

Surface disinfection (wipe disinfection), bactericidal, yeasticidal per VAH [German Association for Applied Hygiene] recommendations: 2.5% (25g/l) – 15 mins.; 2% (20g/l) – 30 mins.; 1% (10g/l) – 60 mins.

### APPLICATION:

Surface disinfection by wiping (recommended procedure): Apply a sufficient quantity of the prepared CyberTech Surface disinfection non-alcoholic solution to the surfaces to be disinfected. Use mechanical action to completely wet all surfaces (scrubwipe disinfection).

### COMPOSITION:

100 g contains: 6.66 g alkyl (C12-16) dimethylbenzyl ammonium chloride; 3.33 g didecylmethyl ammonium chloride; < 5 % non-ionic surfactants, perfumes.

### SPECIAL INFORMATION:

Can cause severe skin burns and eye damage. Very toxic to aquatic life. Wear protective gloves/protective clothing/eye protection/face protection. IF SWALLOWED: rinse mouth. Do NOT induce vomiting. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/ shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. IF exposed or concerned: Call a POISON CENTER or doctor/physician. Avoid release to the environment. Dispose of contents/ container to approved disposal company or local collection.

### PRODUCT STATUS/LISTINGS:

Conform to the EU Biocidal Product Regulation (BPR, 528/2012). Conform to the EU guideline 93/42/EC for medical devices.

### AVAILABLE DELIVERY FORMS

Itemnumber	Packaging
9882791	5L Canister

### AVAILABLE ACCESSORIES:

Itemnumber	Packaging
9796276	Tab/dosing device for 5L canister
9796278	Canister-key
9796272	Dosing pump for 5L Canister

Use disinfectants safely. Always read the label and product information before use.

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