## HANDDISINFECTION GEL

**Alcoholic hand disinfectant gel  
to be rubbed into the skin**

- **Alcohol based**

### PRODUCT DESCRIPTION:

CyberTech Handdisinfection gel is an alcoholic hand disinfectant gel free from cumulative longterm acting ingredients that might have an irritating or sensitising effect on the skin. Precise dosing and easier, non-drip application through a user friendly gel formulation. Compatible with all washing lotions. Free from aldehydes, phenols and quaternary ammonium compounds.

### AREAS OF APPLICATION:

Ready-to-use alcohol disinfectant for hygienic hand disinfection.

### SPECTRUM OF ACTION:

Bactericidal, yeasticidal, tuberculocidal, virucidal (HBV, HIV, HCV).

### APPLICATION TIME:

Hygienic hand disinfection (EN 1500): 3 ml - 30 secs.

### APPLICATION:

For hygienic hand disinfection, rub hands with undiluted solution and keep them damp for 30 seconds.

### COMPOSITION:

100 g contains: 63g ethanol (CAS: 64-17-5)

### SPECIAL INFORMATION:

Highly flammable liquid and vapour. Causes serious eye irritation. Harmful to aquatic life with long lasting effects. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. Dispose of contents/container to approved disposal company or local collection. For Professional Use Only.

### PRODUCT STATUS/LISTINGS:

Conform to the EU Biocidal Product Regulation (BPR, 528/2012).

### AVAILABLE DELIVERY FORMS

Itemnumber	Packaging
9882794	100ml bottle
9882795	300ml dispenser bottle
9882796	500ml Eurodispenser Bottle
9882797	1L Eurodispenser Bottle
9882798	5L Canister

### AVAILABLE ACCESSORIES:

Itemnumber	Packaging
9796274	Dosing pump for 500ml
9796275	Dosing pump for 1L
9796276	Tab/dosing device for 5L canister
9796278	Canister-key
9796272	Dosing pump for 5L Canister

Use disinfectants safely. Always read the label and product information before use.

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Bactericida, levaduricida, tuberculicida, virucida (inclusive VHB, VIH, VHC).

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