# SAFE-ST SURFACE DISINFECTION NON-ALCOHOLIC

- Rapid surface disinfection, alcohol-free
- For disinfecting and cleaning medical inventory, medical devices and all kinds of surfaces
- Particularly recommended for acrylic glass
- Strong cleaning power
- Fast-acting

## PRODUCT DESCRIPTION:

Alcohol-free, ready-to-use solution for disinfecting and cleaning of medical inventory, medical devices and other kinds of surfaces. Particularly recommended for alcohol sensitive medical devices and surfaces. Based on quaternary ammonium compounds. Safe-ST Surface Disinfection non-alcoholic contains no alcohols, aldehydes or phenols.

## AREAS OF APPLICATION:

Acc. to Medical Device Directive (MDD): Disinfection and cleaning of alcohol-sensitive, non-invasive medical devices. Acc. to Biocidal Products Regulation (BPR): For disinfecting and cleaning all types of alcohol-sensitive surfaces

## SPECTRUM OF APPLICATION AND EXPOSURE TIME:

Surface disinfection (wipe disinfection) bactericidal, yeasticidal per VAH [German Association for Applied Hygiene]: 1 min., effective against all enveloped viruses per the RKI recommendation incl. HBV/HIV/HCV/Vaccinia/BVDV/Influenza/flu viruses: 30 secs.

## APPLICATION:

Thoroughly dampen surfaces and objects. After the application time has elapsed, wipe down with a disposable wipe, where necessary.

## COMPOSITION:

100 g ready to use solution contains: 0.5 g Alkyl (C12-16) dimethylbenzyl ammonium chloride; 0.25 g Didecyldimethylammoniumchloride.

## SPECIAL INFORMATION:

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to approved disposal company or local collection.

## PRODUCT STATUS/LISTINGS:

Conform to the EU Biocidal Product Regulation (BPR, 528/2012).

Conform to the EU guideline 93/42/EC for medical devices.

## AVAILABLE DELIVERY FORMS:

Itemnumber	Packaging
9797731	1L Spray bottle
9797732	5L Canister

## AVAILABLE ACCESSORIES:

Itemnumber	Packaging
9796277	Sprayhead for 1 Liter Bottle
9796276	Tab/dosing device for 5L canister
9796278	9796278 Canister-key
9796272	Dosing pump for 5L Canister

Use disinfectants safely. Always read the label and product information before use.

# SAFE-ST SURFACE DISINFECTION NON-ALCOHOLIC

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